DRÄGER Alco-Check 0,5

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7. Ausgabe

Februar 2000

DEUTSCH

Allgemeines

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ENGLISH

General

The alcohol content in exhaled air is dependent on the alcohol content in the blood. If the Alco-Check reveals alcohol in the exhaled air, a subsequent blood-alcohol test or another approved method is essential to establish a legal assessment. No blood-alcohol test is of course necessary when the Alco-Check produces no indication.

Alco-Check contains corrosive substances.

Keep out of the reach of unauthorized persons.

Prerequisites

Testing of the exhaled air for alcohol must be performed no earlier than 15 minutes after

- the consumption of alcoholic or aromatic beverages (e.g. fruit juice),
- use of mouth sprays,
- smoking.


Otherwise an excessively high value is measured.

Alco-Check contains a yellow reaction layer which turns green when exposed to alcohol vapour.

An unused Alco-Check which has turned green before testing must not be used.

An Alco-Check which has been opened but not used, or which may have not changed colour during a test, must not be used again. The unit may be used for only one test.


Measurement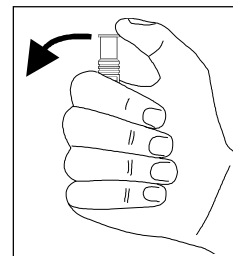

Remove the Alco-Check from the packaging and bend the ends one after another (see figure) until they click.

The Alco-Check is then activated and can be used.

- Insert the Alco-Check into the opening of the flattened measuring bag with the printed arrow pointing towards the bag.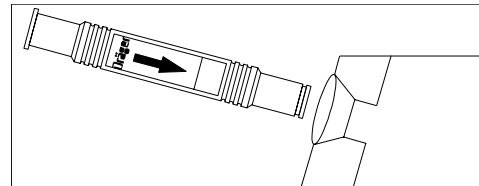


Dräger

- Inhale deeply and blow strongly into the Alco-Check input, filling the bag completely without pausing to inhale again. The blowing should be finished in 15 seconds to 20 seconds.

Evaluation


With a blood-alcohol concentration between 0.3‰ and 0.4‰, the green colour does not reach the indicator line.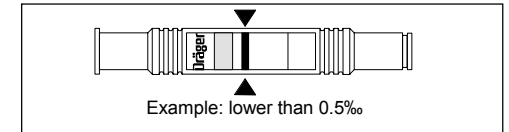

If the green colour reaches or goes further than the indicator line, a blood-alcohol content of 0.5‰ or more can be assumed.

Disposal

Alco-Check contains corrosive substances, and the two ends should therefore be sealed off with the plugs supplied after use.

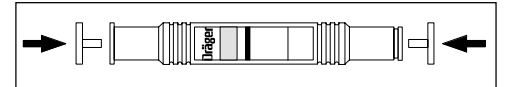Do not simply throw away used Alco-Check or Alco-Check which have reached the end of their shelf life so they do not come into the hands of children.

Liability

Alco-Check can only be used for guideline measurements. **We therefore accept no liability whatsoever.**

Even when used as carefully as possible, there may be deviations between the breath-alcohol concentration and the blood-alcohol concentration which is the sole decisive factor in law.

Our advice: Even blood-alcohol concentrations less than 0.5‰ can affect driving ability. Do not drive under the influence of alcohol.

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